




## POSIZIONAMENTO DEGLI ELETTRODI HEARTVIEW 12L 12 Lead ECG Recorder

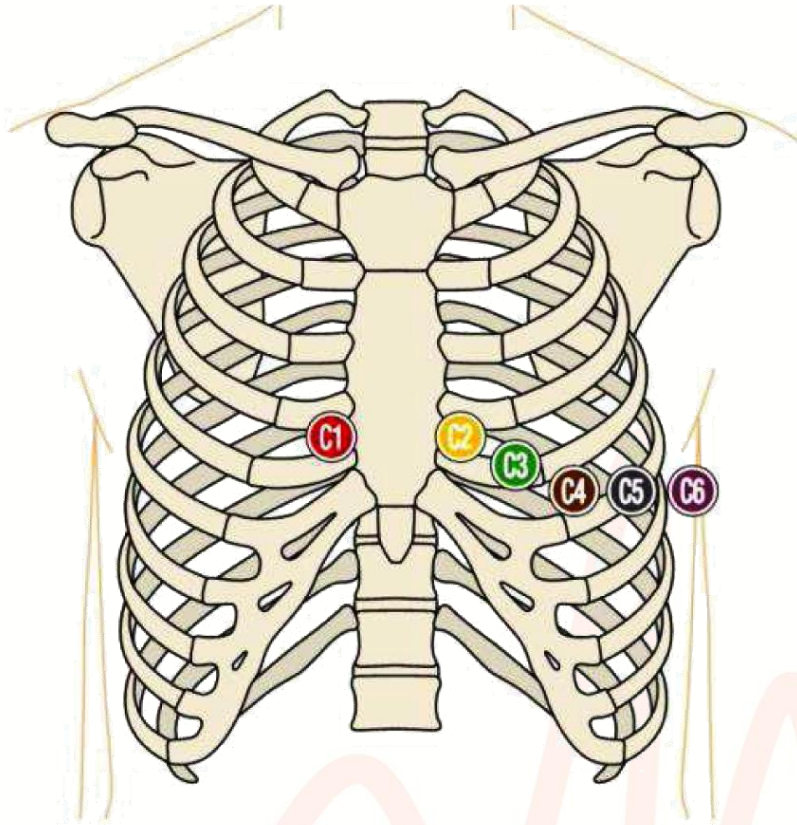

### Limb electrodes

- |  |  |
|--|--|
| <span style="color: red;">●</span> R Right upper arm | <span style="color: yellow;">●</span> L Left upper arm |
| <span style="color: black;">●</span> N Right ankle   | <span style="color: green;">●</span> F Left ankle      |

### Chest electrodes

- C1 4th intercostal space at right sternal edge
- C2 4th intercostal space at left sternal edge
- C3 midway between C2 and C4
- C4 5th intercostal space at left midclavicular line
- C5 same horizontal level as C4 on left anterior axillary line
- C6 same horizontal level as C4 on left midaxillary line